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Listed by FDA

## Medical Face Shield Instruction Manual

- 1. Product name:** Medical Face shield
- 2. Model:** PM-1
- 3. Product description:** The face shield is composed of a protective cover made of polymer material, a foam strip and an elastic band. It is non-sterile, and is for single use.
- 4. Intended use:** The product is worn by the healthcare staff to limit cross operating theatre and care units.
- 5. Operation description:**
  - 5.1 Before use, please make sure this product is clean.
  - 5.2 Please remove the mask double-sided protective film before wearing.
  - 5.3 Bend the mask into an arc, and try it on.
- 6. Warning:**
  - 6.1 Please read the instructions carefully before use.
  - 6.2 This product rarely causes allergic reactions. Once an allergic reaction occurs, please stop using it.
  - 6.3 This product cannot protect people from flame or spark.
  - 6.4 It is forbidden to use this product when it is damaged or deformed. It is forbidden to use when the packaging is damaged.
  - 6.5 Long-term use of the elastic rope may weaken the elasticity, which is normal.
  - 6.6 This product cannot replace closed protective equipment.
- 7. Storage and maintenance:**
  - 7.1 This product should be stored in a room with a relative humidity of not more than 80%, a temperature of -20 °C to + 55 °C, good ventilation and of no corrosive gases.
  - 7.2 Avoid direct sunlight.
  - 7.3 Prevent external squeezing and deformation during storage.

### 8. Marks:

Trademark

Keep dry

Manufacturer

Handle with care

Date of manufacture

Do not re-use

CE marked product

Recovery

Follow instructions for use

Authorised Representative in the EUROPEAN COMMUNITY

**9. Statement:** Refer to the product qualified certification for the production date of the product. Service life: 3 years.

Guilin Woodpecker Medical Instrument Co., Ltd.  
Information Industrial Park, Guilin National High-Tech Zone, Guilin, Guangxi, 541004 P. R. China  
Tel: Europe Sales Dept.: +86-773-5873196  
North/South America & Oceania Sales Dept.: +86-773-5873198  
Asia & Africa Sales Dept.: +86-773-5855350 Fax: +86-773-5822450  
E-mail: woodpecker@glwoodpecker.com, sales@glwoodpecker.com  
Website: http://www.glwoodpecker.com

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Lot number: 20200506

Inspector: 02

Manufacturing address: Information Industrial Park, Guilin National High-Tech Zone, Guilin, Guangxi, 541004 P. R. China

**Guilin Woodpecker Medical  
Instrument Co., Ltd.**

PASS

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MedNet EC-Rep GmbH